





Insert Ultrasonic Unit Cable plug into back of Ultrasonic Unit

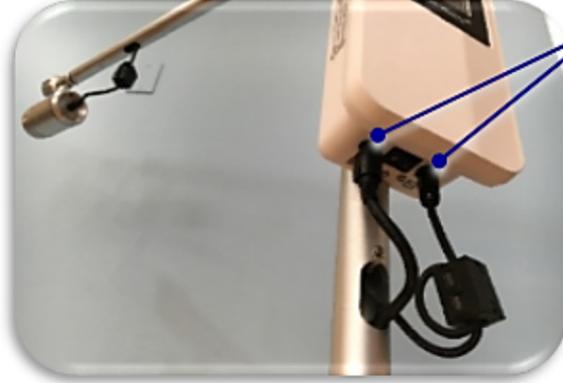

Insert Ultrasonic Unit Cable plug AND Power Cable connector into bottom of Control Unit according to printed graphic.



COMPLETED SYSTEM ASSEMBLY

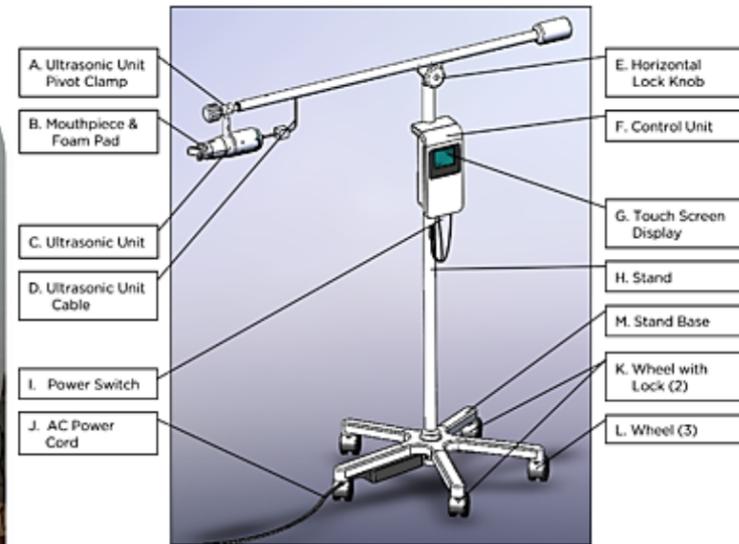

uPhoria System - Part Descriptions



If you have any questions, please call **uPhoria** Global Customer Services Center at 1-617-737-1199.

SYSTEM & STAND BASE BOX – CONTENTS (1 OF 2)



Base
Power Supply
Power Supply Cable



Instructions For Use
Mouthpiece, 2 each
Ultrasonic Unit
AC Power Cord, Power Supply
Control Unit
Tools / Hardware

POLES BOX – CONTENTS (2 OF 2)



Vertical Pole,
Gasket, Lock Nut

Horizontal Pole
UU Pivot Assembly
Stand Pivot Assembly
Weight
Ultrasonic Unit Cable

'A' goes into 'B' (above)
'C' goes into 'D'



- Place Vertical Pole & Mount into base;
- Lay down Vertical Pole/Base with Casters on floor;
- Place 2" O-ring and Tighten Lock Nut




- Insert Power Supply cable up into Vertical Pole
- Withdraw cable connector when it appears in the Vertical Pole opening
- Push up cable completely, and pull all slack out of the Vertical Pole opening



- Insert Ultrasonic Unit Cable and Ground Wire into Vertical Pole
- Pull out both Cable and Ground Wire from opening in the Vertical Pole;
- Place Pivot assembly fully into Vertical Pole (Confirm O-ring is between Pivot and Vertical Pole).



- Install Ground Wire with M3x6mm screw to front of Vertical Pole;
- Place Ferrite Clamp around Ultrasonic Unit Cable loop (Confirm cable passes through Ferrite Clamp **TWICE**).

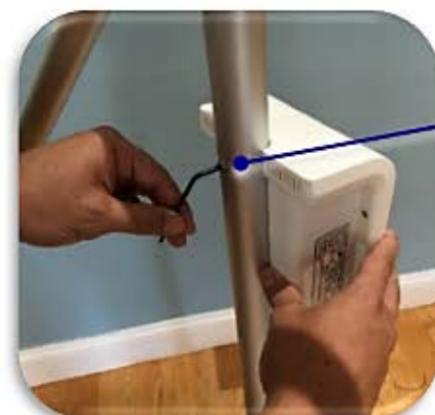

- Align & hold Control Unit on Vertical Pole, Install top Bolt (the shorter one, 35mm) using 2.7mm L-key wrench;
- Install lower Bolt (the longer one, 40mm)



- Loosen Knob (counterclockwise) on Ultrasonic Unit Pivot, then thread the Ball Mount into the Ultrasonic unit
- Tighten the Ball Mount into the Ultrasonic Unit body using the screwdriver
- Tighten the Knob (clockwise)