

# BRIGHTER TEETH IN 4 MINUTES

uPhoria



BEFORE



AFTER




Introducing **uPhoria**, the most technologically advanced in-office whitening system on the market. **uPhoria**, brightens your smile by *4 Shades or more in only 4 Minutes*, making it an effective, convenient, and affordable whitening treatment that won't disrupt your day or your pocketbook.


**uPhoria**, combines the power of ultrasound with cavitation, It activates our proprietary whitening gel through ultrasonic waves, to create the fastest whitening product available at your dental office. The treatment is safe on enamel and soft tissues.

Experience the **uPhoria** of whiter teeth today, and walk out with a healthier, whiter smile. Smile with confidence.

PLEASE CALL DENTOVATIONS GLOBAL CUSTOMER SERVICE CENTER AT:  
617-737-1199 OR VISIT [UPHORIASMILE.COM](http://UPHORIASMILE.COM) FOR MORE INFORMATION

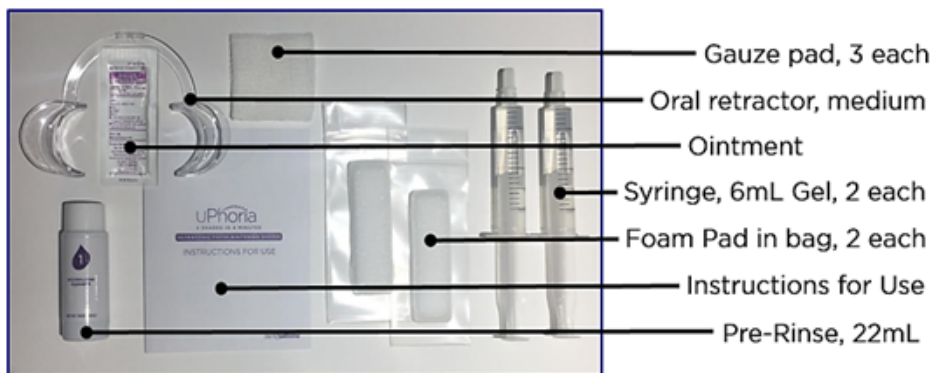

- Gauze pad, 3 each
- Oral retractor, medium
- Ointment
- Syringe, 6mL Gel, 2 each
- Foam Pad in bag, 2 each
- Instructions for Use
- Pre-Rinse, 22mL

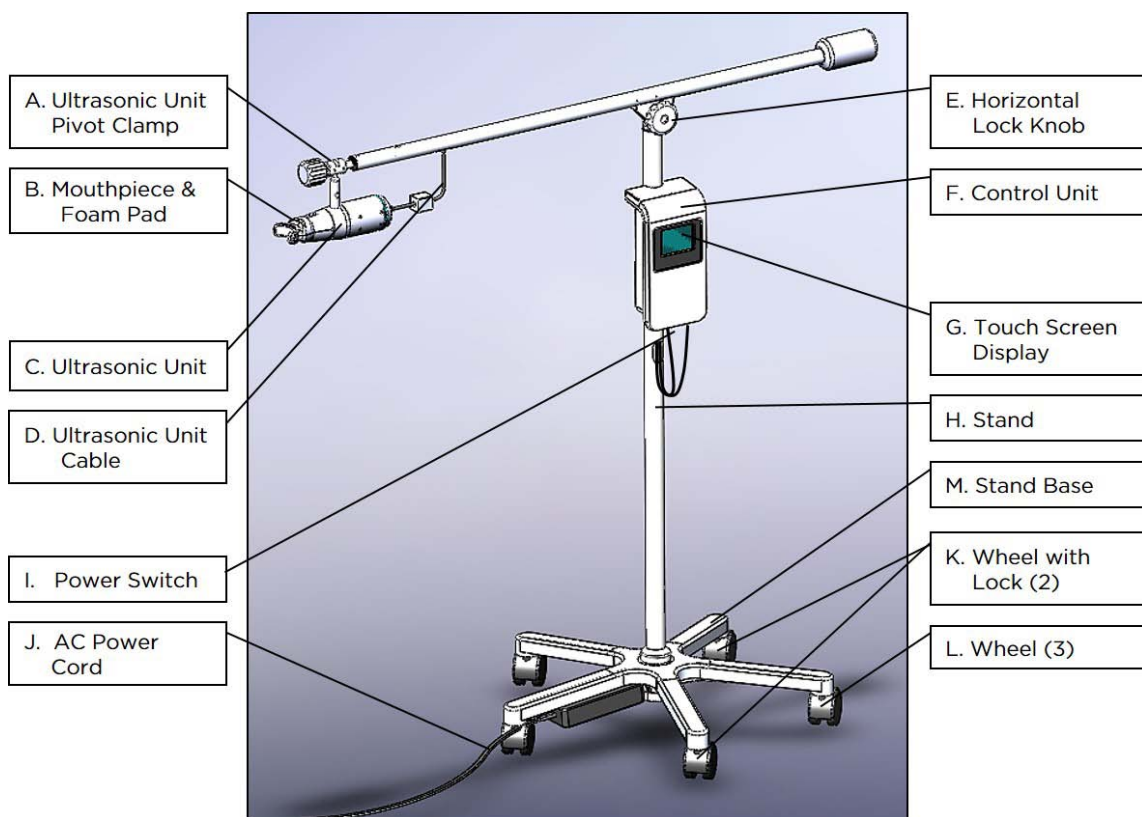

- A. Ultrasonic Unit Pivot Clamp
- B. Mouthpiece & Foam Pad
- C. Ultrasonic Unit
- D. Ultrasonic Unit Cable
- I. Power Switch
- J. AC Power Cord

- E. Horizontal Lock Knob
- F. Control Unit
- G. Touch Screen Display
- H. Stand
- M. Stand Base
- K. Wheel with Lock (2)
- L. Wheel (3)

1. **Determine SHADE** of patient's teeth.
2. **Liberaly apply Vitamin E** (or Petroleum Jelly) to patient's lips.
3. **Rinse** with 10mL of Pre-rinse for 30 seconds (1/2 bottle).
4. **Attach Mouthpiece** onto Ultrasonic Unit - **MUST BE TIGHT**.
5. **Insert Gauze Pads**, place safety glasses and oral retractor.
6. **Soak Foam Pad** with 6mL of Whitening Gel.
7. **Place Foam Pad** onto sterile Mouthpiece by fitting straps behind arch.
8. **Align Ultrasonic Unit/Mouthpiece/Foam Pad** onto teeth
9. **Press START** to begin 2:00 minute treatment countdown. 3 tone chime indicates that treatment is complete.
10. **Move system, and remove first foam pad**; Repeat steps 6 to 9 with 2nd Foam Pad soaked with Gel placed onto Mouthpiece.
11. **Press START** for second 2 minute treatment. Rinse again to refresh.
12. **Determine POST treatment shade**, confirm results.
  - Position Stand, Ultrasonic Unit and Mouthpiece nearby, in advance.
  - Treatment may be paused to reposition Mouthpiece, as needed.
  - At START, blue light illuminates. Normal operation includes a pulsing hiss, a gentle warming, and a tingle feeling.
  - Foam Pad should gently press onto top and bottom teeth evenly. Instruct patient to hold top and bottom teeth in line.